

INSTALLATION GUIDE BATH STOOL

IMPORTANT

- Thoroughly inspect all components for damage or defects prior to use.
- Always check to ensure the stool is stable before each use.

ASSEMBLY INSTRUCTIONS

1. Attach the legs to the cross-frame. Ensure that the frame tabs are firmly slotted into the leg sockets.
2. Place the seat onto the legs and insert each leg into the seat holes.
3. Push down above each leg to ensure the seat is fitted securely—check that the locking tab on the inside is fully engaged (figure 3a).
4. When in use, the seat pad can be rotated by holding the tab upwards while turning. Release the tab to prevent the pad from moving.
5. To adjust the height, press down the locking pin and pull the base of the leg until the pin clicks into the next hole. Repeat until the desired height is achieved. Ensure that all legs are configured at the same height to prevent a non-level seating position.




1



2



3

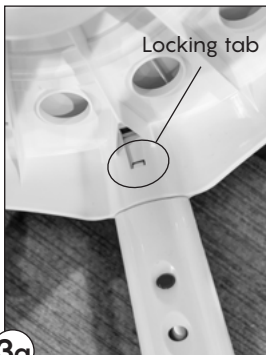

3a



4



5

CARE AND MAINTENANCE

- Clean with a soft cloth and warm soapy water.
- Do not use harsh detergents, corrosive chemicals, or abrasive cleaners.